Opis wyrobu:

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Product description:

Alternating pressure bubble anti-bedsore mattress equipped with end flaps which allows to place the mattress on the bed. Maximum weight capacity: 120kg.

Use:

Prevention and support of bedsore treatment. The mattress is inflated by a system of pipes. The cells are alternately inflated which ensures changing position of body support points. Patient's body rests on the same, invariable height regardless of the phase of pump functioning.

Contraindications:

None

Phases of pump functioning:

- 1 phase: Inflating all the cells
- 2 phase: Deflating the first section of cells
- 3 phase: Reinflating all the cells
- 4 phase: Deflating the second section of cells

The phases occurs in cycles every 6 minutes.

Preparation to use

1. Place the mattress on a standard bed so that the air intake would be located on the side of patient's legs. Put the end flaps under the bed mattress.
2. Check if the switch is in the "off" position.
3. Connect the pump to electricity.
4. Connect the pipes to the pump – insert them into the air outlet located on both sides of pump and on the mattress.
5. Place a pump on hard surface or hang it on hinges. The place where the pump will stand must be clean and dry. Dust, hair or other impurity may block the filter.
6. Please, check if the pipes are not folded or pressed.
7. Switch on the pump.
8. Set the pressure regulator on maximum. It will shorten the time of pumping which usually takes 15-20 min.
9. After inflating the mattress, you should choose the pressure according to the doctor's recommendation.
10. Place a patient on the mattress.

In order to receive exact instructions concerning an appropriate use and optimal pressure, please, consult your doctor. The mattress and pump create effect of "constant movement" (changing position of body support points), provide constant muscle massage and improve blood circulation. Thus, the mattress is an effective tool to prevent bedsore. But the position of the patient on the mattress needs to be changed despite of using it.

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ATTENTION!!!

Please remember that on the mattress should stay only the patient (the other persons on the mattress are not allowed). On inflated mattress you are not allowed to carry the physiotherapy treatment or rehabilitation.

The mattress should be utterly protected against contact with body care cosmetics as it causes a danger of permanent damage of the mattress.

Switching off

1. Turn the switch into the "OFF" position.
2. Disconnect the pipes from the air outlet in order to prevent air from escaping.

Cleaning and storage

Disconnect the pump from electricity. Clean all the surfaces with a moist cloth and gentle detergent. Then, wipe it again with a moist cloth. The pump must be dry before setting ON. The pump can be cleaned ONLY after disconnecting the electricity supply.

ATTENTION !!!!

1. Using the isopropyl alcohol or dissolvent may damage the pump.
2. Please, make sure that the pump is completely dry before setting on.

The mattress should be cleaned with water, soap and gentle detergent. After cleaning, dry it carefully. The mattress mustn't be washed in high temperature, dried in a washing machine, bleached, chemically washed, ironed or pressed.

The mattress should be folded while storing. Before folding, please, check if it is completely dry which prevents it from moulding. The mattress should be stored in a dry place. Do not put the mattress close to the fire.

Problem solving

Do not try to repair the pump, do not open it as it can lead to electric shock!

1. The pump is switch on and does not work.
 - a) Please, check if the cable is connected to electric supply and to the pump. If not, you should connect it.
 - b) Please, check if the fuse has overheated. If so, you should replace the fuse.
 - c) Please, check if there is a voltage in the electric circuit. If there is not, find the cause of such situation.
 - d) Please, check if the cable is not damaged. If so, you should change the cable.
2. The pump works too slowly or the pressure is too low.
 - a) Please, check if the pressure regulator is set on an appropriate pressure. If not, please, regulate it.
 - b) Please, check if the pipes are correctly connected to the air outlet. If not, press them in such a way which prevents the air from escaping.
 - c) Please, check if the pipes are not folded or pressed. If so, please, unbend them.
 - d) Please, check if the cells are not damaged. If so, please, repair it using the reparation kit.

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Wymiary: 28cm x 10,5cm x 9,5cm

Waga: 1,4 kg

- e) Please, check if the pump works correctly – if you can hear some strange sounds like hissing etc., you should submit it to the reparation.

Safety of using

ATTENTION!!!

You mustn't use the mattress close to inflammable materials. It can lead to explosion !

In order to avoid accidents, please follow the below rules:

1. Always check if the pump is connected to an appropriate socket (220V/60 Hz).
2. Always make sure the plug rests outside the socket when the pump does not work.
3. Always make sure the pump rests in a dry place, please, remember that it cannot get wet.
4. Please, do not touch the pump when it is wet. Always disconnect the pump from electricity when it is.

In order to avoid damages and fire:

1. Never leave the working pump unattended.
2. Please, pay special attention if the pump is used in the presence of children or disabled people.
3. The pump can be only used according to its purpose. Do not use it with other, unauthorized devices.
4. Never use the pump with cable or plug, which is damaged or moist. You should submit such a pump to a service point.
5. The cable should be kept away from the hot surfaces.
6. Never leave the pump in a place where the cable or pipes can be dangerous for people who are not familiar with the device.
7. While disconnecting, please, always pull the plug not the cable.
8. Never block the pump filter. The filter must be free from any impurities such as hair, fur etc.
9. Always consult your doctor before using the mattress.

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